

Tekst 11



Fluramon

Instructions for Use

Fluoride varnish for stabilization of dental enamel. **Fluramon** contains highly effective ammonium fluoride and thus acts as prophylaxis against caries and hypersensitive teeth. **Fluramon** is colourless and therefore no discoloration of teeth occurs.

Efficacy:

Fluramon is a fast drying varnish, which adheres well to dry tooth enamel and dentine. This makes it possible to achieve intensive protection of the treated surfaces over a longer period of time, particularly of the sites of predilection of caries. **Fluramon** forms a watertight protective film insulating against thermal and chemical influences. Object of every use of **Fluramon** must be to retain fluoride varnish as long as possible on the treated surfaces.

The application of **Fluramon** is recommended for preventional treatment of hypersensitive necks of teeth and caries. **Fluramon** can particularly be applied at the margins of cavities after grinding of individual teeth, after fillings have been placed, especially when the etching technique has been applied, as well as after insertion of crowns, inlays and bridges.

Surfaces to be treated are cleaned and thoroughly dried with air.

Fluramon must be shaken well before each use. Solid particles must be well dispersed. **Fluramon** to be dropped onto a brush, preferably on **Pele Tim** foam pellets and the surface to be thinly treated.

Attention: Layers, which are too thick, separate too easily. Varnish is to be diluted if necessary.

The varnish is allowed to be absorbed for 10 - 20 seconds and then dried with air.

With correct application and proper mouth hygiene **Fluramon** remains in place for several days.

The application of **Fluramon** can be repeated two or three times at intervals of 7 days. Surfaces should be dried and touched again. **Fluramon** must be as liquid as to run easily from the dropper after shaking. It may be necessary to dilute varnish with a few drops of the solvent provided. This solvent is also useful for cleaning instruments, etc.

Presentation Fluramon:

Set 4 g + solvent	Art. No. 1001
clinic bottle 10 g	Art. No. 1002

Supplementary indications:

Pele Tim foam pellets are the ideal carrier for all liquid and pasty preparations, instead of cotton wool, and preferably used for the application of **Fluramon**.

Presentation Pele Tim:

Size No. 1 small ø 4 mm	approx. 3000 pellets	Art. No. 2252
Size No. 2 medium ø 5 mm	approx. 1000 pellets	Art. No. 2253
Size No. 3 large ø 8 mm	approx. 500 pellets	Art. No. 2254
Size No. 4 v-shape 6 x 8 mm	approx. 500 pellets	Art. No. 2255



P.O. Box 767
Germany
Telex 232 202 dent d

D-27457 Cuxhaven
Phone (47 21) 719-0
Fax (47 21) 719-140

Lees bij de volgende vragen steeds eerst de opgave voordat je de bijbehorende tekst raadpleegt.

■ Tekst 11 Fluramon

- 1p 42 Waarschuwt de bijsluiter bij het middel 'Fluramon' voor een schadelijke bijwerking, en zo ja, welke? Zo nee, antwoord "nee".